

A New Way to Manage Oral Secretions

Continuous oral secretions deserve continuous management



Remove oral secretions *before* they become a problem


Convert to continuous and hands-free oral secretion management.

SWIK® was developed by a patient who knew there had to be a better way to care for patients with dysphagia.



Features & Benefits of SWIK

- ✓ Hands-free
- ✓ Continuous
- ✓ Comfortable
- ✓ Discrete
- ✓ May reduce infection risk
- ✓ Fewer nursing interactions
- ✓ Compatible with ventilated patients
- ✓ Gentle on oral tissues
- ✓ Controls drooling
- ✓ Less medical waste
- ✓ Improved experience
- ✓ Targeted solution
- ✓ Up to 12 hours of control

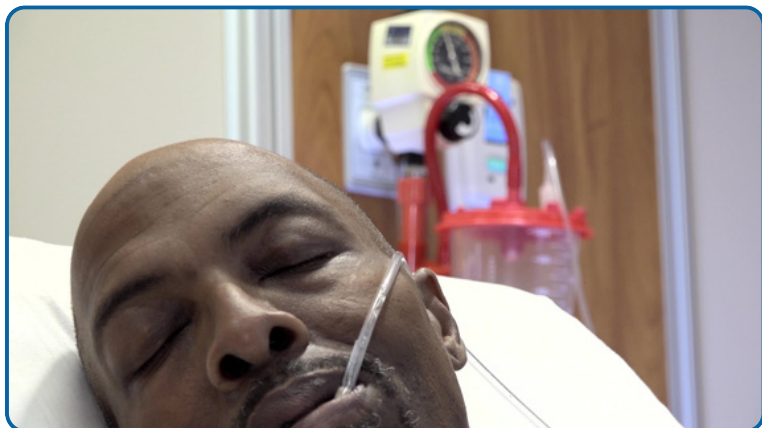

In-patient



At home



On the go

Why does continuous suction matter?

An NIH Study* concluded that continuous clearance of oral secretions, “may have an important role to play in reducing the rate of VAP, decreasing the duration of mechanical ventilation, and shortening the length of stay of patients in the ICU.”

70%

Reduction in VAP
(Ventilator Associated Pneumonia)

51%

Reduction in the
length of ICU stay



46%

Reduction in the
duration of mechanical ventilation



A New Way to Manage Oral Secretions



SWIK is a single-use device intended to continuously manage oral secretions in the mouth without frequent intervention. **SWIK** includes a wicking foam mouthpiece that fits in the patient's bottom gum line and removes secretions in the mouth for up to 12 hours. **SWIK** is connected to tubing and is compatible with portable and wall-mounted pumps.

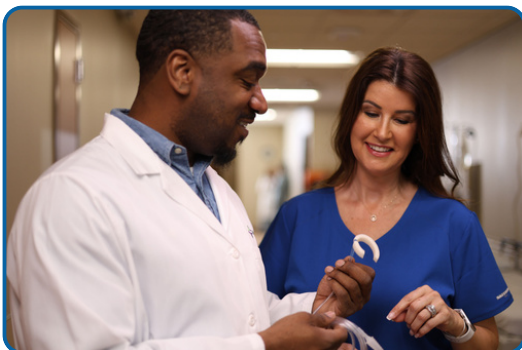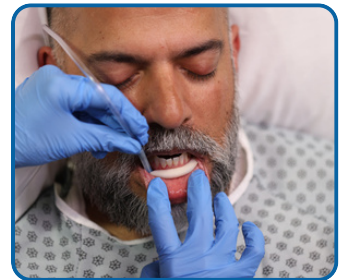

Patients and clinicians agree it is time for a new way to manage oral secretions. A patient-use study conducted in the ICU at a level I trauma center rated **SWIK** as clinically beneficial, safe, and potentially effective in reducing hospital acquired aspiration pneumonia.

Using **SWIK** requires 2 components and a negative pressure pump (not included):

1. **SWIK** mouthpiece with catheter (SWK-M16), used for up to 12 hours
2. **SWIK** Extension Tubing (SWK-T56), used for up to one week at a time

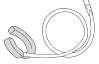Mfr #		Product Description
SWK-M16		SWIK® Mouthpiece and 16" Catheter
SWK-T56		SWIK® 56" Extension Tubing



Watch
SWIK



- Single-use
- Latex free
- DEHP free

Our products are covered by the following patents:

- US Utility Patent 11,744,933
- US Utility Patent Application 18/461,028
- US Design Patent D1,001,267

Medicare HCPCS Codes: A4628, A7002
FDA Registered and Listed
♥ Made with care

Purchasing Information

In-patient / Out-patient

sales@swikmedical.com
swikmedical.com



VA / DOD / IHS

governmentquotes@redonemmedical.com
912-662-1443
FSS#: 36F79720D0236



Provided By:

Medical Dynamics
TECHNOLOGY OF THE FUTURE, TODAY™
866-624-3952
www.thinkMDI.net

Our mission is to improve the lives of patients and caregivers

 swikmedical.com

 info@swikmedical.com